REFILLABLE BODILY FLUID SPILL KITS

BE PREPARED FOR EMERGENCIES WHEREVER YOU ARE



Quickly and safely lock down hazardous bodily fluid spills with refillable bodily fluid spill protection kits. This spill protection kit contains two bodily fluid spill kits in one convenient storage container. When a spill kit is used, simply order a refill rather than reordering a full protection kit.

THE DIFFERENCE IS NIGHT AND DAY.™



REFILLABLE BODILY FLUID SPILL KITS

BE PREPARED FOR EMERGENCIES WHEREVER YOU ARE







Refillable Bodily Fluid Spill Protection Kit

This refillable kit comes with two bodily fluid spill protection kits in one easy storage container. Cut costs by simply ordering a refill pack when needed instead of ordering a whole new kit.

0.00	ID SMIT MOLEC	
	a paperty from all for	
	Married .	Constant
	- market	Standards.
work for Water (P. P.)	a) belowed that with factor	
or is belocious tripped.		
	Quality to Streton P	
of Reproposals partitional		
		rich.
man addited school in		
	successed area with garren	
production states		
	Type sorts per	
	udina whited na-m	Salari squirera
	111	

Description	QTY	Item #
Bodily Fluid Spill Kit	1 kit	114707
Refill Pack	1 pack/refill	114705

Kits Contain

- 1 Red Z Solidifier, 21 gm.
- 2 Pair of Exam Gloves
- 1 Combination Mask/Face Shield
- 3 P.A.W.S. Antiseptic Cleansing Wipes
- 2 Sanizide Plus Wipes (8" x 8")
- 1 Disposable Gown with Sleeves
- 1 Disposable Bonnet
- Shoe Covers
- Scoop/Scraper
- 2 Biohazard Bags with Twist Ties
- 2 Clear Bags with Twist Ties
- 2 Wiper Pads
- Instructions
- Identification Tag

How to use the Bodily Fluid Spill Protection Kit













- 1. Open absorbent powder pack and sprinkle enough powder to evenly cover body fluid incident.
- 2. Using scoop and scraper (and/or disposable towels), pick up solidified material and put into the red biohazard bag.
- 3. Thoroughly spray the approved cleaner/disinfectant over the spill area and allow for at least a 10 minute contact time.
- Use disposable wiping cloth to wipe up all residual disinfectant.
- Place used personal protective equipment (PPE) and waste into the red biohazard bag and then tie shut.

